KATALOG PRODUKTÓW
PRODUCT CATALOGUE

SPIS TREŚCI

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CONTENT

6	INTRODUCTION	42	Silicone pillow (P-SS-21)
	LOWER LIMB STABILIZATION	44	Thermoactive support cushion (P-SS-22)
14	Lower extremity abduction pillow (P-SS-01)	46	Head cushion (P-SS-25)
15	Cylindrical cushion (P-SS-02)	47	Orthopedic neck pillow (P-SS-26)
16	Calf wedge (P-SS-06)	49	Universal cushion (P-SS-31)
17	Knee wedge (P-SS-10)	50	Head immobilizing pillow (P-SS-44)
18	Thermoactive lower limb raising wedge (K-SS-06)	52	Trapezoid headrest (P-SS-55)
19	Thermoactive Braun splint (K-SS-07)		TRUNK STABILIZATION
20	Lower limb raising wedge (K-SS-09)	54	Lateral positioning cushion (P-SS-08)
21	Braun splint (K-SS-10)	56	Lateral positioning wedge (P-SS-12)
22	Thermoactive Braun splint with foot support (K-SS-11)	57	Ring cushion (P-SS-13)
23	Pediatric lower limbs abduction wedge (P-SS-28)	59	Semi-soft bolster pillows (P-SS-14)
24	Universal positioning cushion (P-SS-29)	60	Semi-rigid bolster pillows for trunk support (P-SS-17)
25	Lower limbs positioning wedge (P-SS-30)	61	Pediatric immobilizer (P-SS-23)
26	Lower limbs positioning cushion (P-SS-33)	62	Anti-bedsore donut cushion (P-SS-24)
28	Long lower limb separator (K-SS-31)	63	Reflux baby wedge (P-SS-27)
29	Universal limb separator (P-SS-51)	64	Sensory bean bag floor cushion (P-SS-38)
30	Anti-bedsore cushion for the lower limbs (P-SS-52)	65	Rehabilitation roller (W-SS-01)
31	Cylindrical cushion with the abduction wedge and side supports (P-SS-32)		POSITIONING
	UPPER LIMB STABILIZATION	68	Shaped wedge (K-SS-08)
34	Upper extremity support with protector (P-SS-20)	69	Repositioning cushion (P-SS-54)
35	Thermoactive elevation arm wedge (K-SS-01)	70	Wedge with self-gripping function (K-SS-02)
	HEAD STABILIZATION		ACCESSORIES
38	Anti bedsore round cushion (P-SS-09)	72	Self-adhesive sheet (B-SS)
39	'V' pillow (P-SS-15)		MODULAR MATTRESS
40	Headrest (P-SS-18)	74	Modular mattress (M-SS)
41	Neck ring cushion (P-SS-19)	76	Lower limb cushion with wedge (PM-SS-56)

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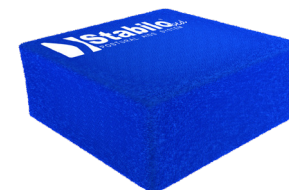
POSTURAL AIDS SYSTEM IN THE LYING POSITION

STABILObed® is a modular system used for precise and effective positioning of the user in various lying positions. **STABILObed®** has a wide range of applications. The system has been registered as a **1st CLASS MEDICAL SYSTEM**, and each product has a **CE** mark. The **STABILObed®** system includes such independent elements as:

Cushions in different sizes.

The **STABILObed®** system consists of cushions and pillows which have been designed to stabilize and position the selected parts of the user's body. They have a washable cover made of an innovative fabric called **PureMed™**. This fabric is safe for people and very easy to disinfect. Individual elements of the system can be used in the diagnostic imaging because they do not contain any metal elements. Depending on the kind, shape and purpose, the **STABILObed®** elements have different fillings.:

Granules – the product has been filled with very light and small polystyrene granules. The granules may move freely inside the product, and as a result it is much easier to adjust the product to the user's body shape. Additionally, the air, which is inside **PureMed™** the cushion, can easily migrate through the vapour-permeable cover around the user's body.

VISmemo™ foam – it is a memory foam which adjusts its shape to the shape of the user's body under pressure. Then, it becomes even softer due to the temperature of the user's body. The **VISmemo™** foam is the most active in the areas with the greatest bedsores risk. As a result, it is the highest standard in the anti-bedsores prophylaxis.

Comfort foam – soft foam which has a lot of small, closed air bubbles. Because of that, it easily adjust to the user's body shape.

Off-loading foam – it is a rigid technical foam, created during the recycling process. The structure of this foam is closed which means that it goes back to its initial shape even after long-term lying of the user.

Silicone – this filling consists of silicone fibers in the round shape with **0,5** cm diameter. The air in the fibers provides great ventilation, preventing sweating. The cushion perfectly adapts to the anatomical shape of the user's body, it is very soft and light. It is pressure-resistant even under the influence of a long-term pressure.

Self-adhesive elastic band sheet - the sheet is made of soft, white or blue velvet. This fabric has a special foam layer which prevents the sheet from rolling on the mattress. Additionally, thanks to the self-adhesive function, it is possible to attach the **STABILObed®** products to it, which prevents them from moving, as most of the **STABILObed®** products are equipped with a set of Velcro. The sheet is elastic so it will not slip out of the mattress. It fits the standard hospital mattresses and is available in three different sizes. There is also a possibility to order a custom size sheet.

INDICATIONS FOR USING THE STABIObed® SYSTEM

- ✓ As an immobilization in diagnostic imaging (MRI, CT scan, X-ray);
- ✓ With bedridden patients who suffer from orthopedic, neurological and rheumatic diseases;
- ✓ With intensive care patients;
- ✓ During rehabilitation, palliative care and positional therapy;
- ✓ During long-term care;
- ✓ As a post-operative immobilization;
- ✓ In a situation when individual parts of the body need relief;
- ✓ In the anti-bedsore prophylaxis.

PureMed™ ADVANTAGES

The stabilizing and positioning **STABIObed®** system is made of an innovative grey fabric **PureMed™** which has numerous advantages:

- ✓ It is safe for people and very easy to disinfect;
- ✓ Ideal for use in areas with the high risk of infection;
- ✓ There is a possibility of multiple use, it does not rub off;
- ✓ It is perfect when the cushions are used by different users, e.g. in hospitals or rehabilitation centers;
- ✓ It is lightweight, soft, elastic and easy to shape;
- ✓ Contains antibacterial components based on inorganic **Silver Zeolite**;
- ✓ It meets all the essential requirements of the medical products of different kinds*;
- ✓ It has **OEKO-TEX® STANDARD 100** certification which is granted to exceptionally skin-friendly parameters;
- ✓ It is used as a cover for hospital mattresses, operating tables and couches;
- ✓ It is waterproof and vapour-permeable in one way only which means that the air from the inside of the cushion may migrate freely outside through the **PureMed™** cover. At the same time, liquids such as sweat, blood or urine will not be able to get through the cover inside the cushion;
- ✓ The **PureMed™** cover has passed the **CIGARETTE TEST** and the **MATCHSTICK TEST** positively, which proves that it is a non-flammable fabric;
- ✓ Complies with **REACH** regulation aimed at ensuring a high level of health and environmental protection.

* According to the Regulation of the Ministers of Health on 3 November 2004 . and the requirements contained in Annex I to Council Directive 93/42 / EEC of 14 June 1993 concerning medical devices , deployed by the law of 30 April 2004 of Medical Devices (OJ L 93 of 2004 . , pos. 896).

PureMed™ TECHNICAL SPECIFICATIONS


Weight:	55 ± 10 [g/m ²]	Tensile strength across:	≥10 [daN/5cm]
Type of PU coating:	100 [%]	Waterproofness:	≥2000 [mm H ₂ O]
PES backing type:	100 [%]	Water vapour permability:	≥300 [g/m ² /24h]
Flammability:	Flame retardant product	Standard width:	145 [cm]
Tensile strength along:	≥20 [daN/5cm]	Standard roll size:	50 [m]

The fabric is certified by National Institute of Public Health and National Institute of Hygiene

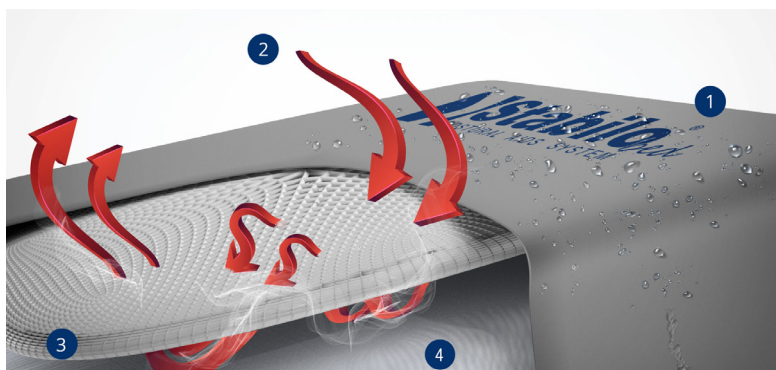HOW IT WORKS?

1. **PureMed™** – a flexible fabric which is safe for people. It is antibacterial, waterproof, vapour-permeable and easy to disinfect.
2. The air inside the cushion can migrate freely through the spacer fabric **AirMesh™** (pt. 3) and around the user's body. Additionally, the **PureMed™** cover will stop any liquids such as sweat, blood or urine.
3. **AirMesh™** – is a special aerating-massaging fabric used in the places, which have a direct contact with the user's body. As a result, the comfort of use has significantly increased, and at the same time the risk of bedsores has considerably decreased.
4. Product filling.

STABILObed® ADVANTAGES

PureMed™ - a fabric which is very safe for people. It is antibacterial and easy to disinfect. Due to the fact that it contains antibacterial components based on **Silver Zeolite**, **PureMed™** fabric is perfect for multiple use by different users in hospitals or rehabilitation centers as well as in cases of a high risk of infection. **PureMed™** is light, soft and easy to shape. This material complies with the **REACH** regulation.

RadioLogic™ - the components of the **STABILObed®** system can be used in the diagnostic imaging (**MRI, CT scan, X-ray**).

VelcroFit™ - the **STABILObed®** elements are equipped with a set of Velcro which allows to position the elements on the mattress covered with a self-adhesive sheet as well as combine selected elements in various configurations.

SkinFriendly™ - this **STABILObed®** component is covered by soft, skin-friendly material.

Velcro Strap System™ - available version with attachment system to use with the modular mattress.

VISmemo™ - it is a memory foam which adjusts its shape to the shape of the user's body under pressure. Then, it becomes even softer due to the temperature of the user's body. The **VISmemo™** foam is the most active in the areas with the greatest bedsores risk. As a result, it is the highest standard in the anti-bedsores prophylaxis.

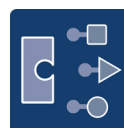
AntySlip™ - a special non-slip fabric which prevents the cushions and pillows from moving on the mattress.

EasyFit™ - a special system of adjustments helps to shape the **STABILObed®** elements and adjust them to the user's needs.

Pocket closed with the use of Velcro - you can easily put the proper amount of the granules depending on the individual needs.

Ready for MODULAR MATTRESS SYSTEM™ - Product compatible with modular mattress M-SS .

MAINTENANCE

PureMed™ should be cleaned according to the laundry guide. Coated materials should be cleaned with the use of neutral **pH** soap solution (about **7**) applied with a soft sponge or a damp cloth. Rub gently and wipe dry to remove the rest of the detergent. In order to keep the original colour and shine level, the procedure should be repeated every month.

Stains should come out if you use **25% solution of ethanol**. Rub gently and wipe dry to remove the rest of the detergent.

Please, bear in mind that some stains might not be completely removed from the cleaned surface. It depends on several factors like active dyes permanently migrating to the material surface, the material composition and finish, or chemical composition of dirt as well as the time the soiling has remained on the material surface.

You should pay particular attention to the generally available domestic cleansers whose components might react with the soiling and cause an effect contrary to what was originally intended.

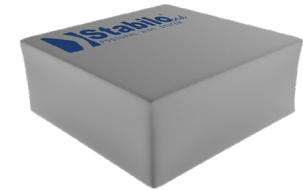
Remember that **PureMed™** should never be cleaned with domestic cleansers and cleaning agents that are generally recommended for 'eco-leather'.

OPTIONAL ACCESSORIES

We have extended our offer of **STABILObed®** through creating very useful optional accessories and equipment, increasing comfort and safety of **STABILObed®** elements. Each accessory has individual code in the catalogue to improve ordering. This code is placed under the size table of every **STABILObed®** product.


PureMed™ washable cover

A washable cover is the outer layer of all the **STABILObed®** cushions. It is made of soft, resilient and easy to disinfect fabric **PureMed™**. The shape of the cover is identical with the shape of a given cushion. Additionally, it has a zip which allows easy removal and putting on the cover. Each cover has a permanent pictogram applied, which presents the safety rules and the ways of disinfecting the cover. In case the cover of your **STABILObed®** damages, there is no need to purchase a new cushion but only replace the cover.


Cotton bag with granules

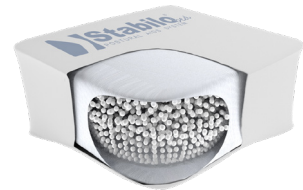There are special cotton bags inside of most of the **STABILObed®** cushions. They are filled with granules which perfectly adapt to the user's anatomy. The bag is made of cotton fabric which is certified by **OEKO-TEX® STANDARD 100** and the granules have obtained the **PZH** certificate. Thanks to a special pocket closed with the use of Velcro you can easily put the proper amount of the granules depending on the individual needs. If, during prolonged use, your **STABILObed®** cushion loses its properties, there is no need to purchase a new cushion but only replace the cotton bag with granules.

Terry cover

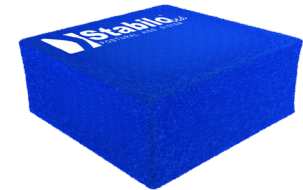The Terry Cover, which is made of soft, cotton fabric friendly for the skin of the user, is certified by **OEKO-TEX® STANDARD 100**. The shape of the terry cover is identical with the shape of a given cushion, it is dark blue and has a fastening which makes it easy to take off for washing. The terry cover should be treated as an individual product and be used by one user.

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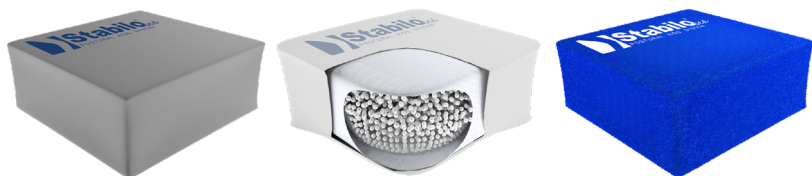
DESCRIPTION

- ✓ The lower extremity abduction pillow protects the user from the adduction contractures of the hips and decreases the risk of bedsores that can appear on the medial side of the lower limbs.
- ✓ It helps to maintain the correct supine position by controlling the legs position, preventing them from crossing.

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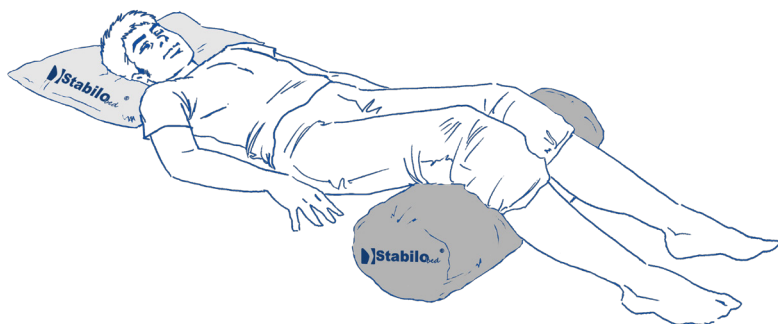
DESCRIPTION

- ✓ The cylindrical cushion can be used in numerous ways, e.g. when located under the knees, it will help in arranging the lower limbs in the Fowler's, Semi-Fowler's or supine position.
- ✓ It is highly recommended for bedridden users.
- ✓ Using the cushion reduces the overstraining of the lumbar spine, improves the circulation and reduces the risk of the swelling of the lower limbs.

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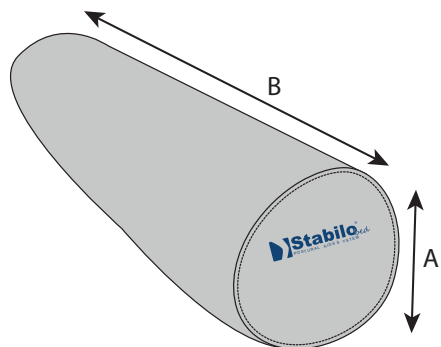
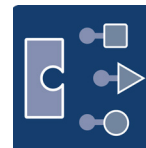
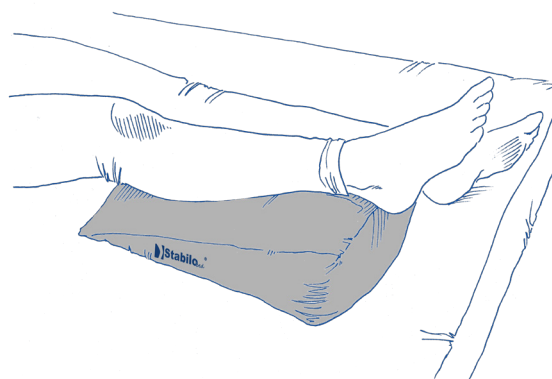
DESCRIPTION

- ✓ It prevents the users from the bedsores that can appear in the heel bone area.
- ✓ Also, it reduces the risk of thrombophlebitis (vein inflammation).
- ✓ The length of the wedge is proper to prevent the knee hyperextension.
- ✓ The **STABILObed®** system is universal so the wedge can be successfully used by users with lower and upper limb injuries as it has relieving properties and guarantees the highest comfort of use.

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DESCRIPTION

- ✓ Using the knee wedge in the supine position reduces the pressure and pain in the lower back.
- ✓ Used together with the **ring cushion (P-SS-13)** helps to put the user in the **Semi-Fowler's** position, where the pelvis and lower back are at **30°**.
- ✓ The knee wedge improves blood circulation and reduces the risk of the lower limbs swelling.
- ✓ Also, it prevents the knee hyperextension.

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
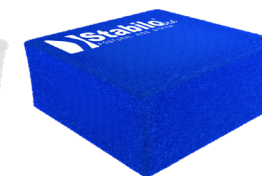
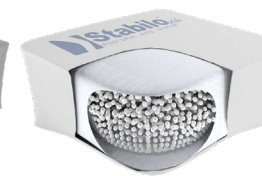
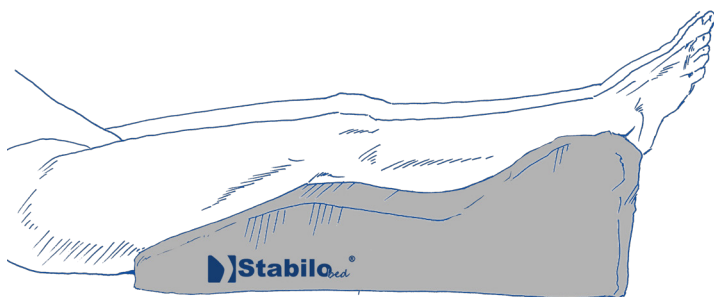
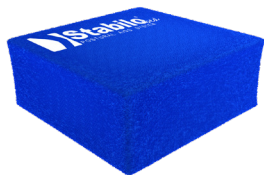
DESCRIPTION

- ✓ The wedge is made of the original **VISmemo™** memory foam.
- ✓ It keeps the lower limbs in the slightly raised position, which improves the blood circulation and reduces the risk of swollen ankles. It can be used as a support during the compression therapy and the anti-bedsore treatment.
- ✓ It is recommended to use the wedge at night because it helps the blood to flow from the legs to the heart which reduces the lower extremity circumference and as a result it is much easier to put the compression stockings on in the morning.
- ✓ The excellent anatomical shape of the wedge prevents the lower limbs deformation.
- ✓ It keeps the pelvis slightly tilted which decreases the pressure and reduces the pain of the lower parts of the spine.
- ✓ The **CertiPur** Certificate means that the foam is strictly controlled in terms of the amount of chemicals, the method of production and the safety of usage. The certificate guarantees that the product is free of heavy metals, phthalates, allergic and carcinogenic substances.

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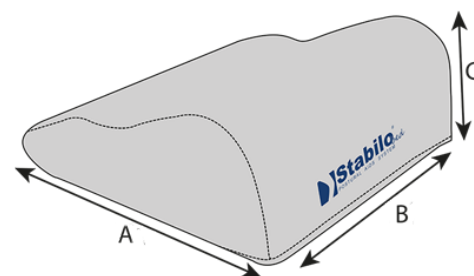DESCRIPTION

- ✓ The Braun splint is made of original memory foam **VISmemo™**.
- ✓ It maintains the lower limb slightly raised, which improves blood circulation and reduces the risk of swollen ankle. It can be used as a support for compression therapy and anti-decubitus treatment.
- ✓ It prevents from bedsores that can appear on heel bone.
- ✓ Reduces risk of thrombophlebitis (vein inflammation).
- ✓ The length is excellent to support the knee against the hyperextension.
- ✓ Our thermoactive splint is anatomically-shaped, which prevents against the lower limb deformations.

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until stocks last. In sizes 0 and 1*, the cushion does not have AntySlip™ material.

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
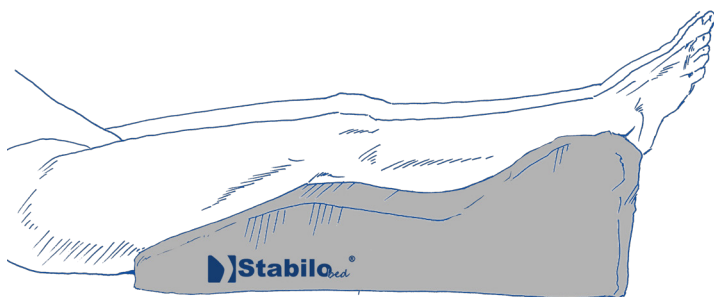
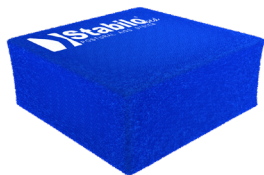
DESCRIPTION


- ✓ It is made of soft foam with numerous closed air bubbles. As a result, the cushion is susceptible to pressure and perfectly adjusts to the shape of the user's body.
- ✓ It maintains the lower limbs slightly raised, which improves blood circulation and reduces the risk of swollen ankles. It can be used as an support for compression therapy and anti-decubitus treatment.
- ✓ It is recommended to use the wedge at night because it helps the blood to flow from the legs to the heart which reduces the lower extremity circumference and as a result it is much easier to put the compression stockings on in the morning.
- ✓ Our thermoactive wedge is anatomically-shaped, which prevents against the lower limb deformations.
- ✓ It maintains pelvic tilt and provides zero gravity back pain relief.

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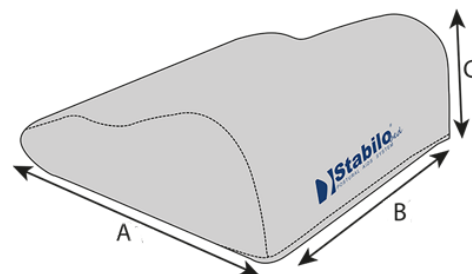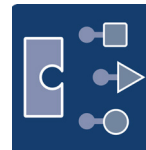
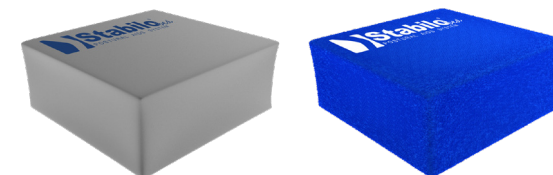
DESCRIPTION

- ✓ It is made of soft foam with numerous closed air bubbles. As a result, the cushion is susceptible to pressure and perfectly adjusts to the shape of the user's body.
- ✓ It maintains the lower limb slightly raised, which improves blood circulation and reduces the risk of swollen ankle. It can be used as a support for compression therapy and anti-decubitus treatment.
- ✓ It prevents from bedsores that can appear on heel bone.
- ✓ Reduces risk of thrombophlebitis (vein inflammation).
- ✓ The length is excellent to support the knee against the hyperextension.

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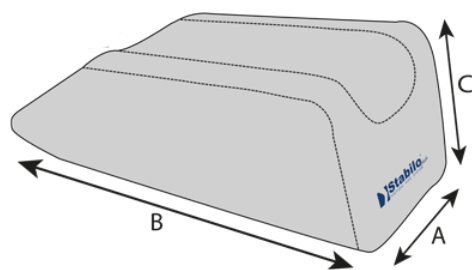
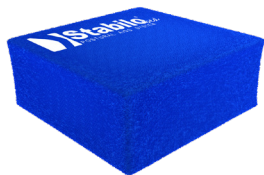
DESCRIPTION

- ✓ The Braun splint is made of original memory foam **VISmemo™**.
- ✓ It maintains the lower limb slightly raised, which improves blood circulation and reduces the risk of swollen ankle. It can be used as a support for compression therapy and anti-decubitus treatment.
- ✓ It prevents from bedsores that can appear on heel bone.
- ✓ Reduces risk of thrombophlebitis (vein inflammation).
- ✓ The length is excellent to support the knee against the hyperextension.
- ✓ Our thermoactive splint is anatomically-shaped, which prevents against the lower limb deformations.
- ✓ The **CertiPur** Certificate means that the foam is strictly controlled in terms of the amount of chemicals, the method of production and the safety of usage. The certificate guarantees that the product is free of heavy metals, phthalates, allergic and carcinogenic substances.

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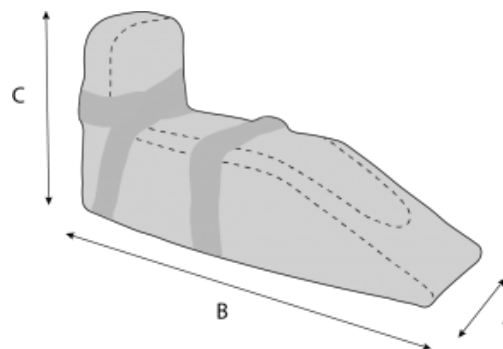DESCRIPTION

- ✓ Pediatric lower limbs abduction wedge protects the young users against the adduction contractures of the hip joints and reduces the risk of bedsores on the medial side of the lower limbs.
- ✓ It helps to maintain the correct supine position by controlling the position of the legs.
- ✓ In order to prevent leg crossing, the wedge has been equipped with a special upper cover fastened with **VelcroFit™**.

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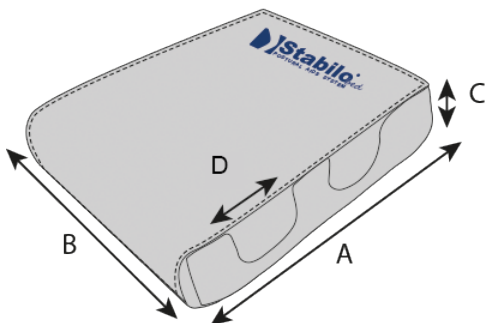
DESCRIPTION

- ✓ The cushion is a perfect addition in post-operative therapy.
- ✓ Its size and appropriate hardness helps in proper support of different body parts, e.g. a foot in numerous therapeutic positions.
- ✓ The cover is made of user-friendly and easy to disinfect **PureMed™** material and **AntySlip™** material, also used in the cover, which prevents the cushion from moving.

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DESCRIPTION

- ✓ Thanks to its design, the wedge enables the correct positioning of the lower limbs in the bends, at the same protecting the user from the adduction contractures in the hip joints.
- ✓ Using the cushion prevents the knee hyperextension in the knee joints and reduces the risk of bedsores on the inner parts of the legs. It is highly recommended for the bedridden users.
- ✓ It can be successfully used in both the supine and semi-reclining position, e.g. in post-operative therapy.

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

DESCRIPTION

- ✓ Lower limbs positioning cushion P-SS-33 is a modern solution in anti-bedsore prophylaxis.
- ✓ Its modular structure provides support and helps in positioning the limbs in bed, in a supine, reclining and sitting position.
- ✓ The cushion has connected segments filled with high-quality granules. As a result it can be easily and quickly adjusted to the user's body, depending on the needs. Using the cushion can successfully relieve the heel area, secure the feet, e.g. against the contact with the bed frame, separate the legs to prevent excessive pressure on the inside of the legs or to prevent their crossing.

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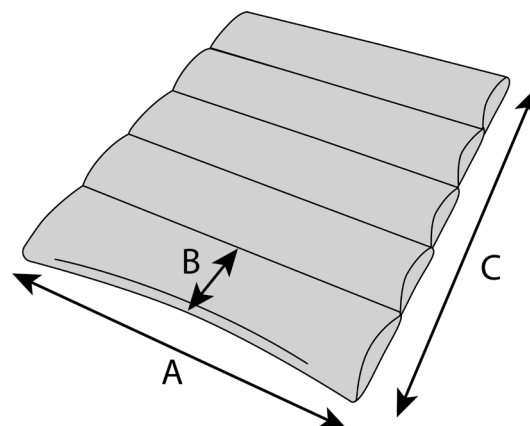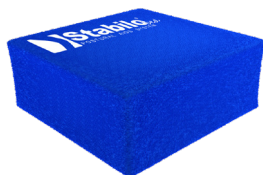
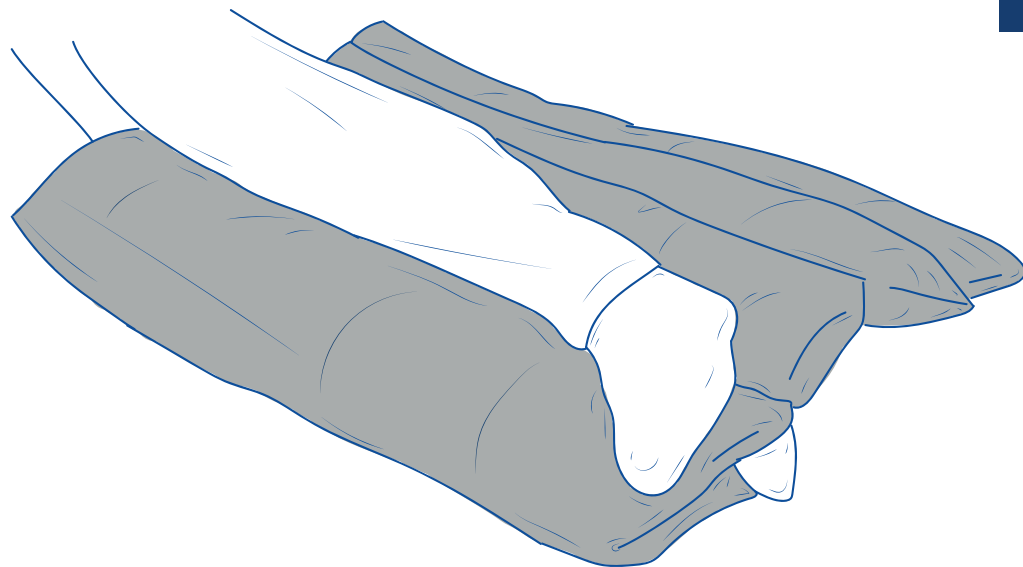
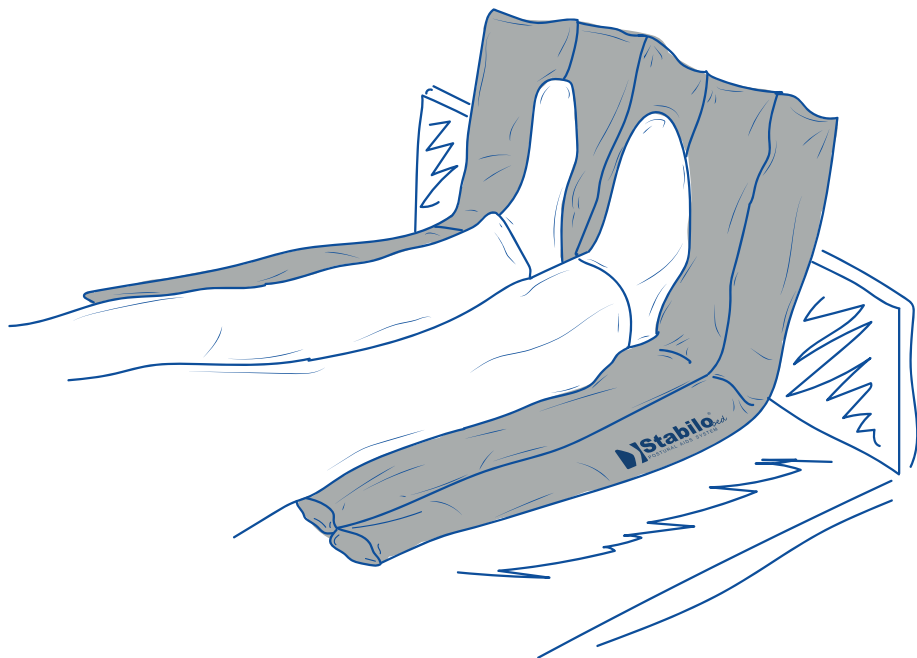
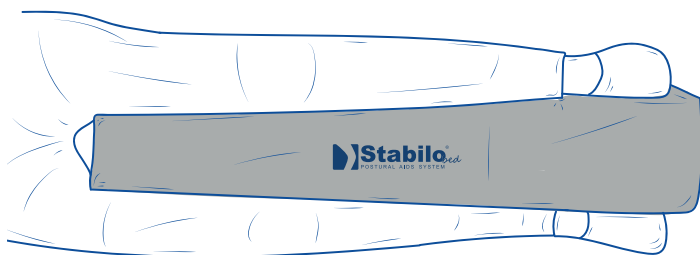
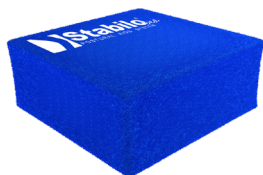
DESCRIPTION

- ✓ The long lower limb separator K-SS-31 protects the user from the adduction contractures of the hip joints and decreases the risk of bedsores that can appear on the medial side of the lower limbs.
- ✓ It helps to maintain the correct supine position by controlling the legs position, preventing them from crossing.
- ✓ The longer shape of the separator gives a much larger contact surface between the body and the cushion, which guarantees much better pressure distribution. The thermo-active, soft VISmemo™ memory foam used in the cushion provides additional comfort of use and anti-bedsores prophylaxis. The foam helps to reduce the risk of pressure injuries or pressure ulcers in particularly sensitive areas such as ankles or the inside of knee joints.
- ✓ The long lower limb separator K-SS-31 is a recommended solution for people staying in the supine or side position for a long time (e.g. bedridden people or people after lower limb surgery especially during the recovery period).

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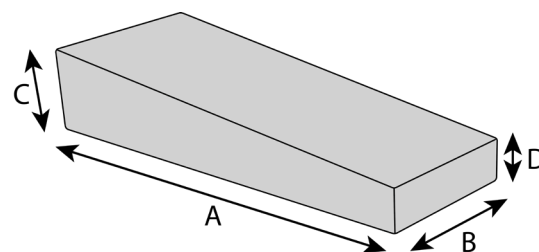DESCRIPTION

- ✓ Thanks to its construction, the cushion enables the correct positioning of the lower limbs in the bends, at the same time protecting the user from the adduction in the hip joints. The side supports placed on both sides of the wedge prevent the legs from falling sideways.
- ✓ Using the wedge prevents the hyperextension in the knee joints and reduces the risk of bedsores on the inside of the legs.
- ✓ It is recommended for bed-bound patients. It can be successfully used in lying and semi-reclining position, e.g. in post-operative therapy.

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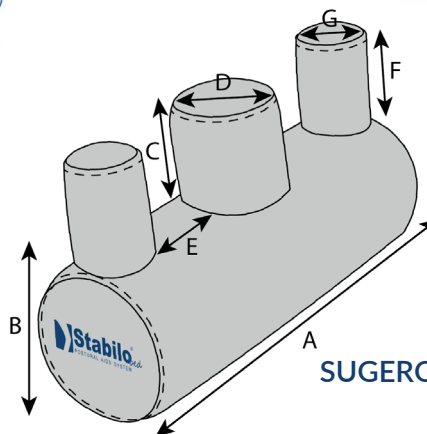
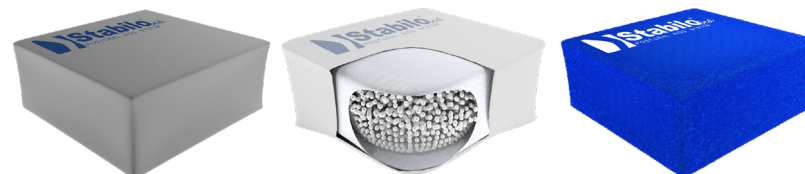
DESCRIPTION

- ✓ Universal limb separator protects the user from the adduction contractures and bedsores. It helps to maintain control over the correct and safe position of the secured limb.
- ✓ Velcro tape allows you to adjust the cushion to the circumference of the limb. Soft foam filling guarantees the highest comfort of use.

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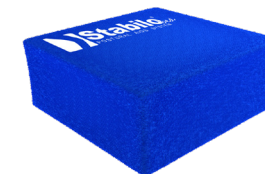
ANTI-BEDSORE CUSHION FOR THE LOWER LIMBS

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
DESCRIPTION

- ✓ Anti-bedsore cushion for the lower limbs P-SS-52 is an excellent choice for people who require additional comfort in the lying position. Its well thought-out design and anatomical shape make the cushion perfectly adjust to the body shape in two ways. First of all, it supports the shins and relieves the area of the heels and ankles while lying on the back. Also, it separates the lower limbs while lying on the side.
- ✓ The thermo-active, soft VISmemo™ memory foam used in the cushion provides additional comfort of use and anti-bedsore prophylaxis. The foam helps to reduce the risk of pressure injuries or pressure ulcers in particularly sensitive areas such as ankles or the inside of knee joints.
- ✓ Anti-bedsore cushion for the lower limbs P-SS-52 is a recommended solution for people with circulatory problems in the lower limbs and for people staying in the lying position for a long time (e.g. bedridden people or people after lower limb surgery especially during the recovery period).

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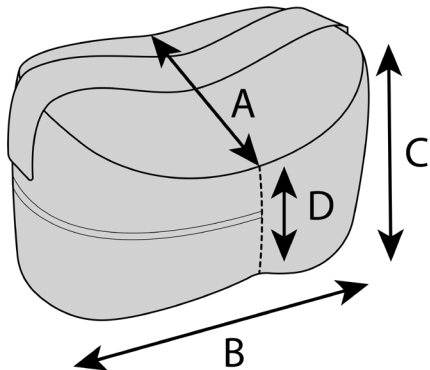
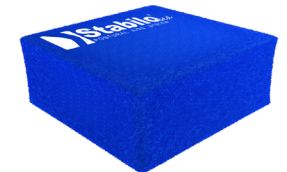
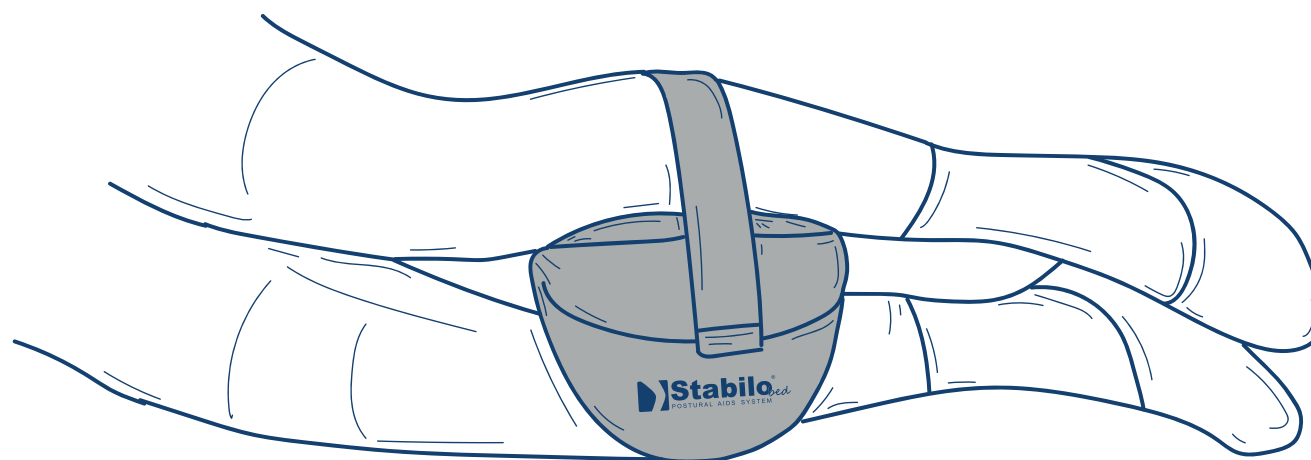
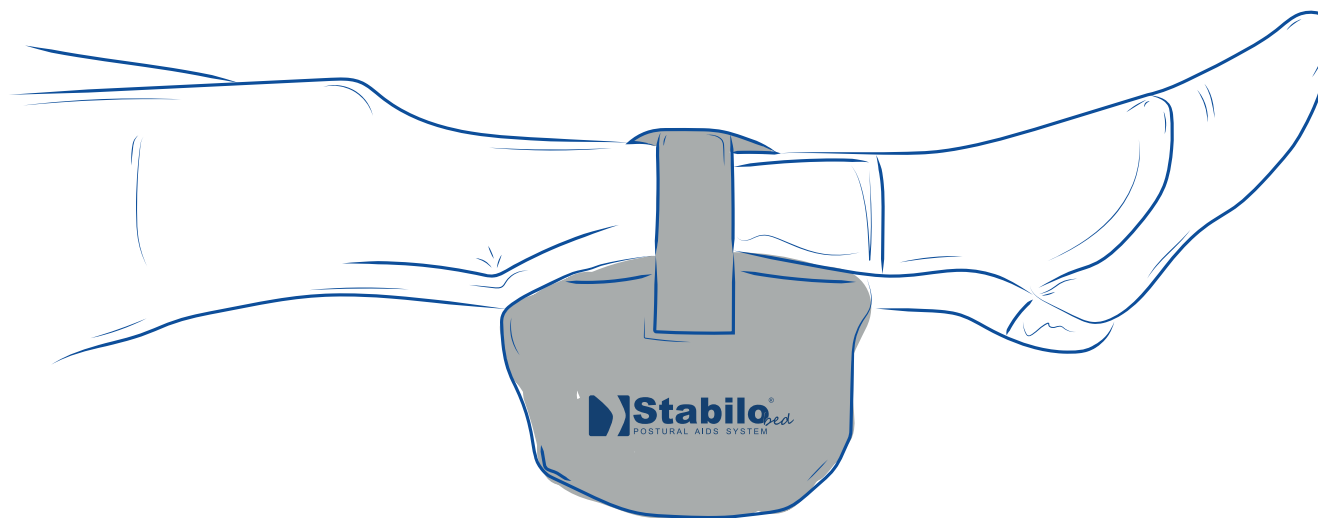
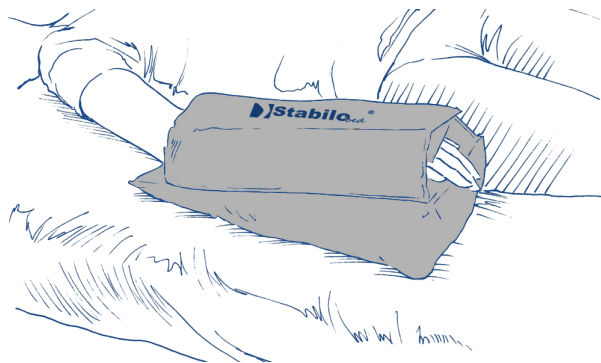
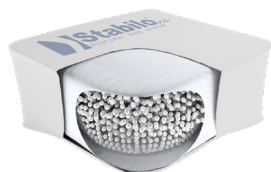
DESCRIPTION

- ✓ The support has been created to protect the upper limb after injuries and surgical procedures. It is equipped with the granules bag which makes it possible to adjust the shape of the cushion to the forearm.
- ✓ It relieves the user's forearm from the fingers to the elbow area, making it ideal for use in anti-bedsore prophylaxis, at the same time preventing the contractures.
- ✓ A specially designed Velcro fastener secures the forearm from the top and reduces its mobility, supporting the healing process.

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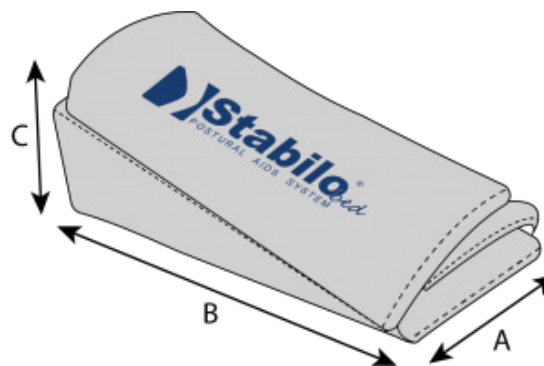DESCRIPTION

- ✓ The elevation arm wedge stabilizes the arm in the slightly raised position, which allows for reduction of the swelling within the distal parts of the limb.
- ✓ It is equipped with the original **VISMemo™** memory foam which reduces the pressure and therefore protects the user against the bedsores in the elbow and forearm areas.
- ✓ The wedge is especially recommended to women after the mastectomy or other procedures where the axillary lymph nodes were excised.

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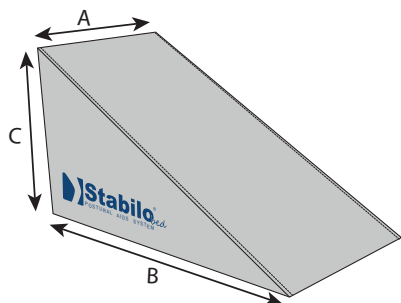
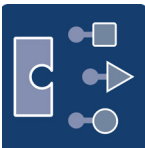
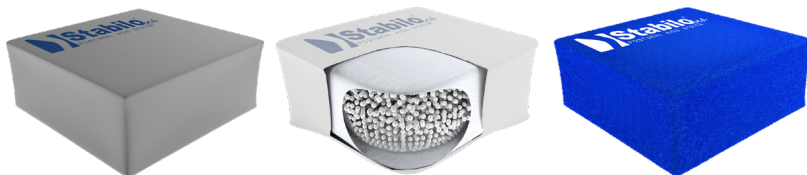
DESCRIPTION

- ✓ The anti-bed sore round cushion can be used in the anti-bed sore prophylaxis, during the pressure ulcers treatment as well as in post-operative and post-traumatic treatment.
- ✓ When used under the head, it prevents the bedsores in the occipital bone area and supports the cervical lordosis.
- ✓ The inner hole in the cushion helps to relieve the sensitive points of the user's body. They do not have a direct contact with the bed which improves the blood supply and nourishment of soft tissues.

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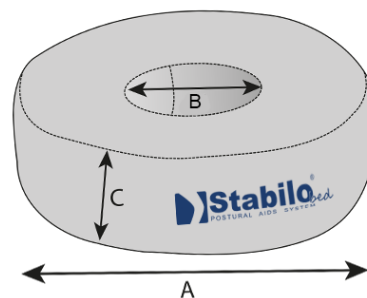
DESCRIPTION

- ✓ The 'V' pillow provides proper support of the cervical spine.
- ✓ It relieves the head and neck and that is why it is often used in the anti-bedsore prophylaxis.
- ✓ It is perfect in the sitting, semi-reclining or supine position.
- ✓ The pillow can be used together with the thermoactive support cushion (P-SS-22).

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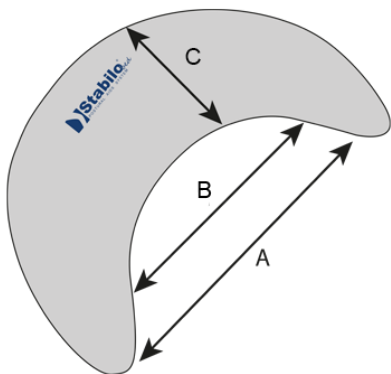
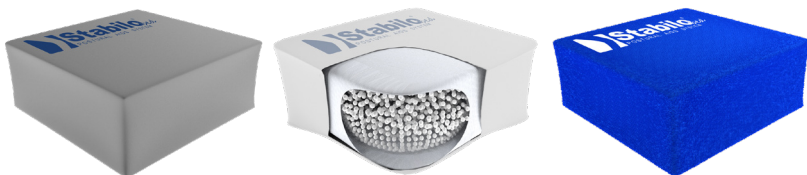
DESCRIPTION

- ✓ The headrest makes it possible to place the lying users in the proper position.
- ✓ It prevents the cervical spinal deformations and can be used in the anti-bedsore prophylaxis.
- ✓ The U shape of the headrest and the granule filling perfectly relieve the head and neck, supporting them in an extremely comfortable way. Thanks to its design, it reduces the tension of the neck muscles.
- ✓ For extra comfort, an optional cotton cover can be used, which is easy to put on and clean.

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DESCRIPTION

- ✓ The neck ring cushion allows to place the head in the middle position both in a sitting, semi-reclining or supine position.
- ✓ It supports the cervical spine and prevents neck deformations.
- ✓ It has a relieving and anti-bedsore effect, protecting the user against the bedsores in the occipital bone.
- ✓ The cushion is equipped with a special Velcro fastening which provides even more effective side stabilization of the head.

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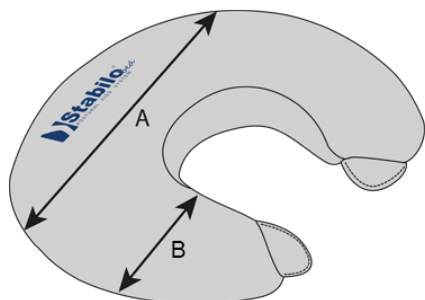
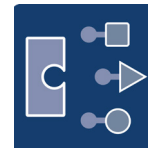
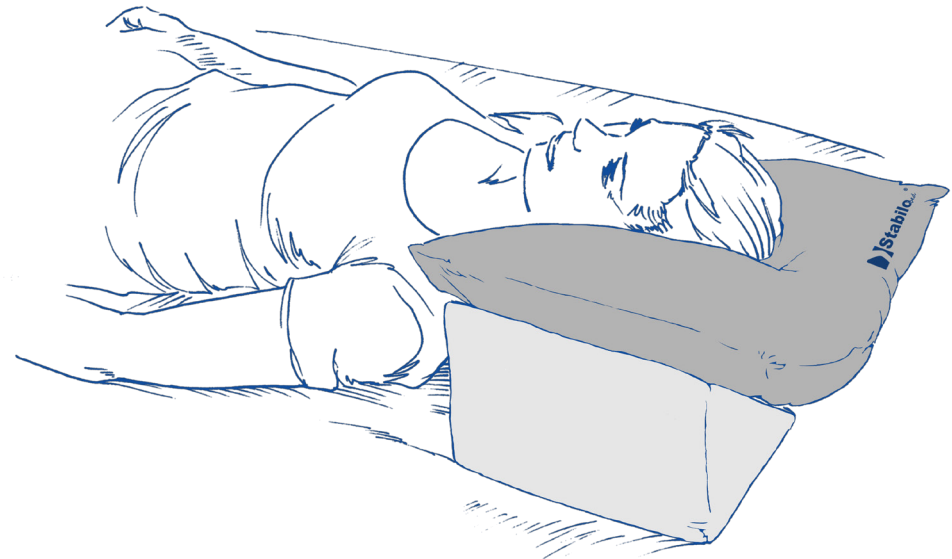
DESCRIPTION

- ✓ It prevents the pathological deformities and neck muscle contractures.
- ✓ The pillow filling guarantees the highest comfort of use.
- ✓ The pillow has a soft, skin-friendly white terry cover.
- ✓ The **STABILObed®** system is universal, that is why the pillow can also be used for supporting the limbs in the supine position or as a separator used between knees or ankle joints in the lateral position.
- ✓ To place the user in the lateral, semi-lateral or supine position, it is recommended to use the **STABILObed®** pillows together with the self-adhesive wedges (K-SS-02) or shaped wedges (K-SS-08).

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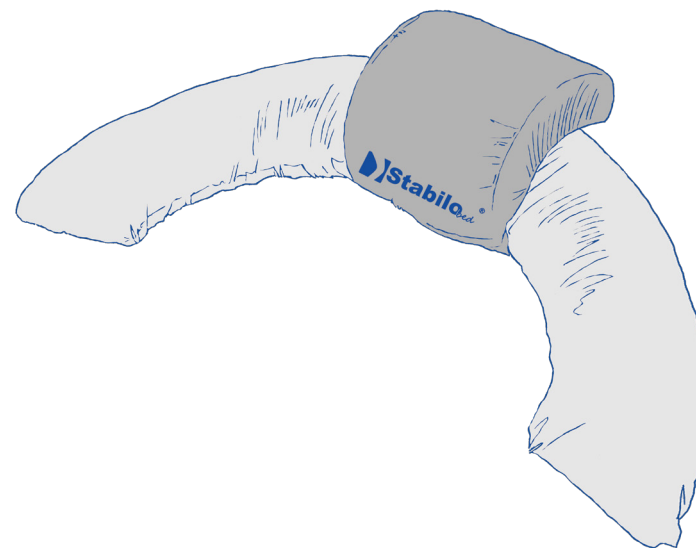
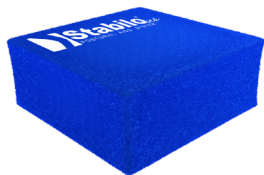
DESCRIPTION

- ✓ The thermoactive support cushion is made of the **VISmemo™** memory foam which perfectly adjusts to the user's body in response to the heat and pressure and as a result it evenly distributes the body weight. After use, the foam returns to its original shape. This solution allows excellent oxygenation and proper blood supply in soft tissues.
- ✓ It is highly recommended in the anti-bedsore prophylaxis so it should be used when the pressure on soft tissues and the risk of bedsores are the highest.
- ✓ The cushion is equipped with a special stabilizing Velcro, which prevents it from moving.
- ✓ Also, it is available with other **STABILObed®** products.

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DESCRIPTION

- ✓ Thanks to its design, the cushion provides an additional side support for the head, and the filling with the thermoactive shape memory foam **VISmemo™** is a guarantee of the highest comfort while lying down.
- ✓ The cushion reduces the pressure in the occiput area, which significantly decreases the risk of bedsores.
- ✓ Also, it is equipped with a special Velcro strip to prevent the cushion from slipping during using.
- ✓ In order to keep the proper hygiene of the product, it is recommended to use an additional **terry cover**.

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DESCRIPTION


- ✓ This pillow has been designed and made as an anatomically profiled 3D pillow. It is made of memory shape foam **VISmemo™** which adjusts perfectly to the user's body shape. Thanks to its design, it provides a proper support for the neck and head.
- ✓ The thermoactive shape memory foam keeps the shape of the body for a short time after we stop putting pressure on it and slowly returns to its original form.
- ✓ The cushion is equipped with a cover made of special **PureMed™** fabric, which is very easy to disinfect.

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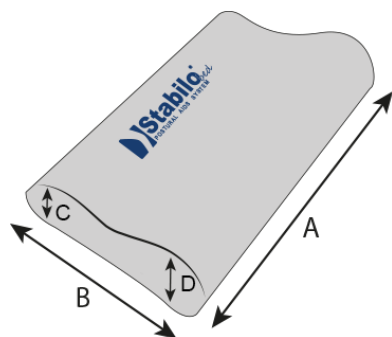
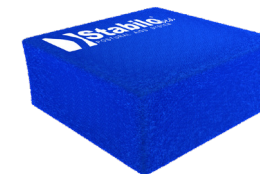
DESCRIPTION

- ✓ A cushion with an anatomic neck cut stabilizes the head in the proper position as well as fills the cervical lordosis and prevents the bedsores around the occiput area.
- ✓ The versatility of the cushion makes it possible to keep the user in a neutral position and reduces the spinal pain.
- ✓ When used between the legs, it protects the user against the bedsores around knees and ankles.
- ✓ Using the cushion also reduces the risk of a pathological adduction, e.g. after the endoprosthetic hip replacement.
- ✓ The cushion has anti-bedsores and stabilizing functions. It can be used in numerous ways, such as:
 - an under head cushion,
 - a cushion used between or under the lower limbs,
 - a support for upper limb elevation (anti-swelling and anti-bedsores function),
 - an aid in maintaining the semi-lateral and semi-recumbent position on the stomach.
- ✓ In order to maintain a proper hygiene of the product, it is recommended to use an additional **terry cover**.

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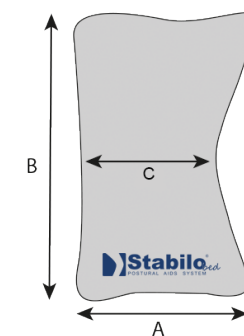
DESCRIPTION

- ✓ The Stabilo P-SS-44 head immobilizer pillow provides safe immobilization of the head and cervical spine when the user is unable to maintain the position independently. It is a medical product that enables the user to maintain the required head position in the event that the head needs to be immobilized (suspected fractures), as well as for specialized examinations such as magnetic resonance imaging (MRI) or computed tomography (CT) scans.
- ✓ The P-SS-44 head immobilizer pillow is made of easy to clean and disinfect materials. It has no aluminum components in its design, making it suitable for use during specialist imaging examinations, such as CT or MRI scans. In addition, the ease of hygiene allows the product to be used by many patients in health and rehabilitation centers.
- ✓ Head immobilizer P-SS-44 consists of several compatible components: a rigid base, an occipital pressure relieving cushion for increased user comfort, 2 side positioning wedges and head stabilization straps with pressure relieving cushions.

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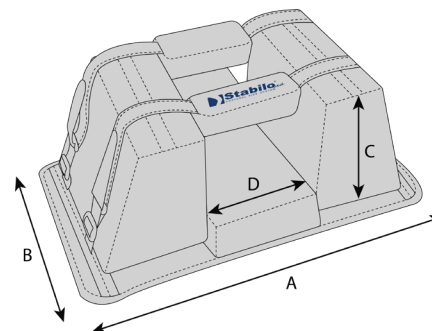
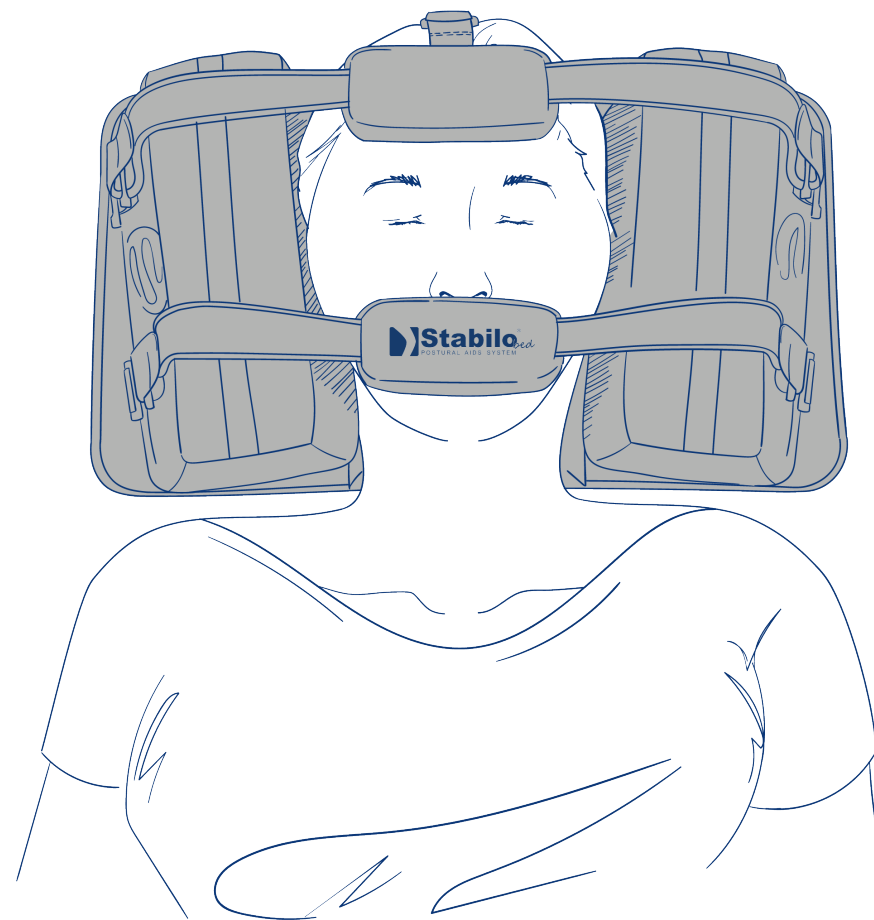
DESCRIPTION

Trapezoid headrest **P-SS-55** provides proper support for the cervical area of the spine. It helps to prevent bedsores as it comfortably and safely relieves pressure on the head and neck thanks to the **VISmemo** memory foam. It is perfect in the sitting, semi-reclining or supine position. It supports and protects the cervical spine from deformations resulting from improper support. The inner hole in the headrest effectively protects the head. As a result, the skin is properly circulated and nourished as it is not in direct contact with the surface. The unique shape of the **P-SS-55** is designed to provide stability and comfort when using the headrest. It also provides pressure relief and support for pressure-sensitive areas on the body, such as the elbow or heel bump area. **P-SS-55** headrest cover is made of soft and elastic **PureMed™** material. It is entirely antibacterial, water-resistant and easy to disinfect, so it can be successfully used by many people, also in such places as hospitals, rehabilitation centers and nursing homes.

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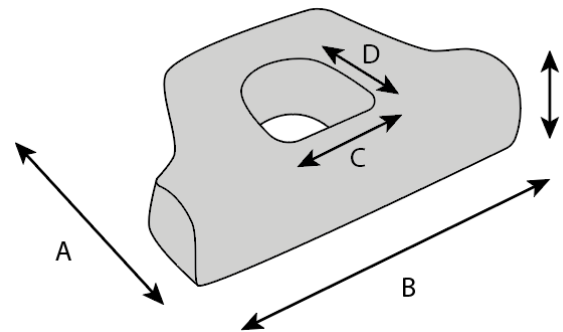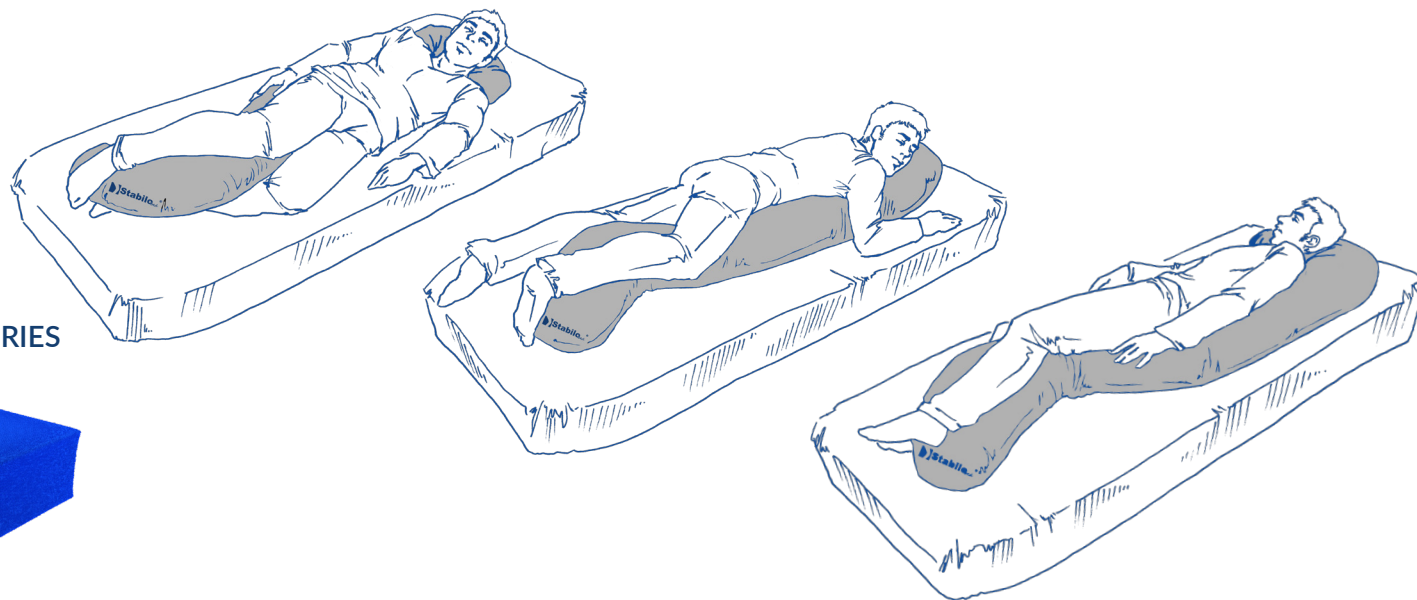
DESCRIPTION


- ✓ The cushion keeps the user in the lateral ('recovery') position and supports knees, pelvic and the upper body.
- ✓ It helps in placing the users in the semi-reclining or semi-prone position.
- ✓ The cushion can be used together with the **thermoactive support cushion (P-SS-22)**, which will be an additional protection against bedsores, while providing even greater comfort for head support.
- ✓ Highly recommended for pregnant women in order to keep a safe side position during the night.
- ✓ Available in one size, might be used by all users regardless of their height.
- ✓ There is a special string and ribbing placed in the cushion, which helps in adjusting the cushion to the required position.

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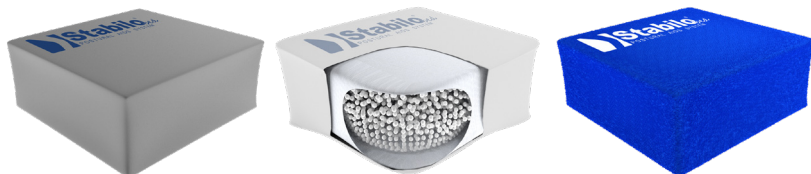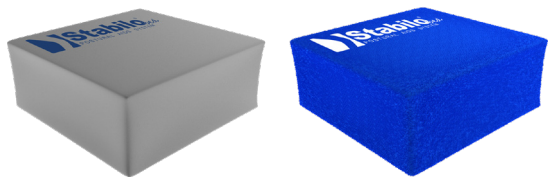
DESCRIPTION

- ✓ Highly recommended for bedridden users who require frequent position changes.
- ✓ Provides comfortable trunk support from one side (left or right).
- ✓ Thanks to the use of the thermoactive **VISmemo™** foam, the wedge relieves the sensitive areas of the body and significantly reduces the risk of bedsores.

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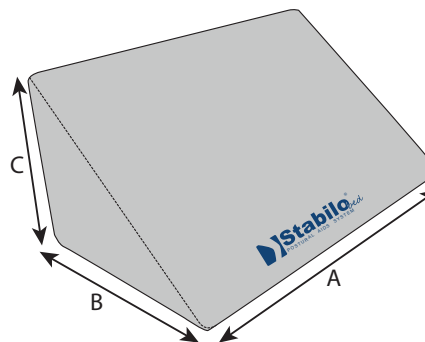
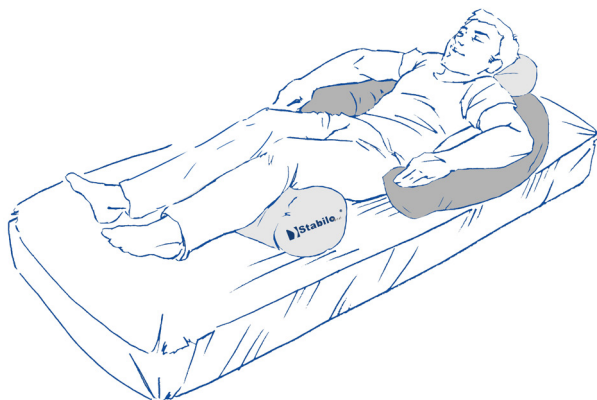
DESCRIPTION

- ✓ Long and bent ring cushion is ideal for users who require the upper body stabilization in the lying position.
- ✓ It helps in proper head positioning and is a great support in the anti-bedsore prophylaxis by relieving the sensitive areas of the body.
- ✓ It can be used by pregnant women to increase the comfort of lying.
- ✓ There is also a possibility to purchase the thermoactive **support cushion (P-SS-22)** which can be used together with the ring cushion.
- ✓ By using the cushion with the **cylindrical cushion (P-SS-02)** or the **knee wedge (P-SS-10)**, we can easily put the user in the **Semi-Fowler's** position.
- ✓ For proper stabilization and high comfort while using the cushion, it has been divided into three parts. This way the right amount of filling can be adjusted on the sides and in the center independently.
- ✓ There is a special string with a ribbing **EasyFit™** which makes it easier to put the cushion in the required position.

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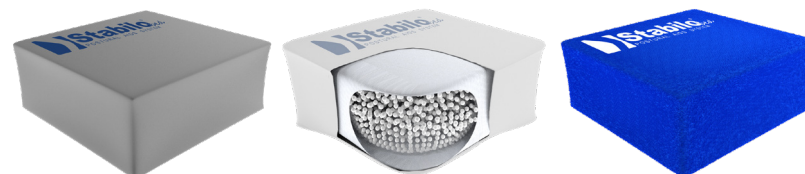
DESCRIPTION

- ✓ The semi-soft bolster pillows are the perfect solution when positioning the users in the semi-lateral or semi-prone position.
- ✓ Highly recommended for pregnant women in order to keep a safe side position during the night.
- ✓ It is a great support in the anti-bedsore prophylaxis.
- ✓ Using the semi-soft bolster pillows helps to prevent the limbs deformation and muscle contractures.
- ✓ To improve the anti-bedsore prophylaxis, it is highly recommended to use the pillow together with the **thermoactive support cushion (P-SS-22)**, which will significantly increase the comfort of supporting the head and neck area at the same time.
- ✓ In order to put the user in the lateral, semi-lateral or supine position, it is advised to use the **STABILObed®** pillows together with the wedge with **self-gripping function (K-SS-02)** or **the shaped wedge (K-SS-08)**.

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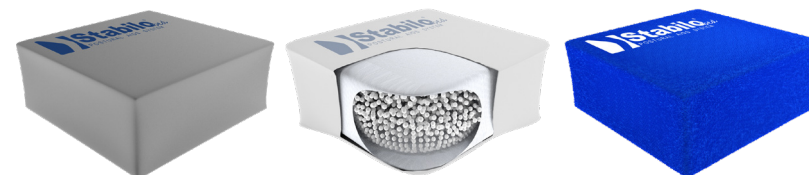
DESCRIPTION

- ✓ The semi-rigid bolster pillows are the perfect solution when placing the user in the lateral, semi-lateral or semi-prone position.
- ✓ They support the anti-bedsore prophylaxis and make the post-operative therapy much easier.
- ✓ For the correct positioning of the user in the lateral, semi-lateral or supine position, it is recommended to use **the shaped wedge (K-SS-08)** together with the pillows.

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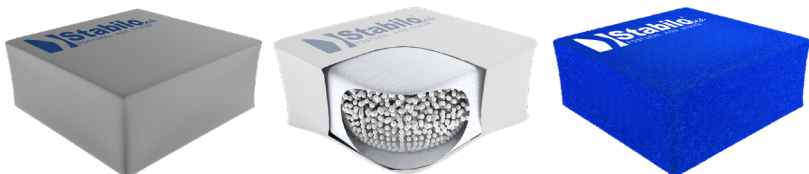
DESCRIPTION

- ✓ The pediatric immobilizer is designed to protect children with the limb and trunk deformities as well as with the spasticity.
- ✓ Using the immobilizer prevents the deformities from deepening and helps in the process of physiotherapy.
- ✓ It can also be used to transport children after injuries.
- ✓ The immobilizer consists of three elements: **the head stabilizer** and **two independent elements with Velcro straps**.
- ✓ Immobilizer does not have any metal elements so it can be used in the **X-ray, MRI and CT** scanning procedures.

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
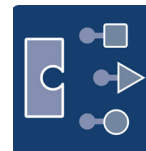
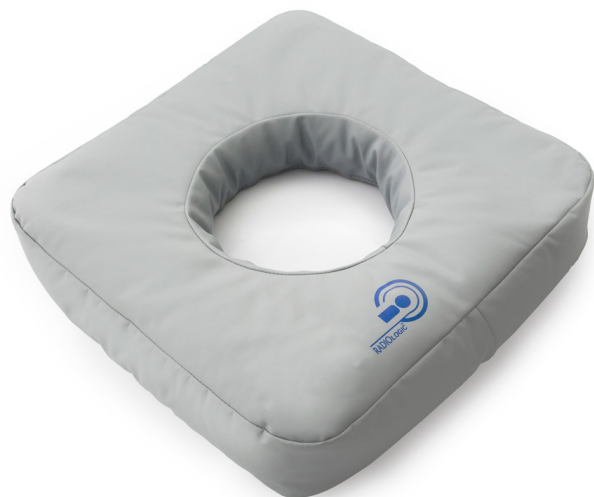
DESCRIPTION

- ✓ This anti-bedsores cushion is used in the anti-bedsores prophylaxis and treatment of the existing bedsores as well as in post-operative and post-traumatic conditions.
- ✓ When used on the seat of a stroller or a car, it protects the user against the bedsores in the area of the ischial tubers and provides an adequate distribution of body weight.
- ✓ The inner hole in the cushion effectively relieves the sensitive points of the user's body, which are properly nourished and supplied with blood when not having a direct contact with the seat.
- ✓ In order to maintain a proper hygiene of the product, it is recommended to use an additional **terry cover**.

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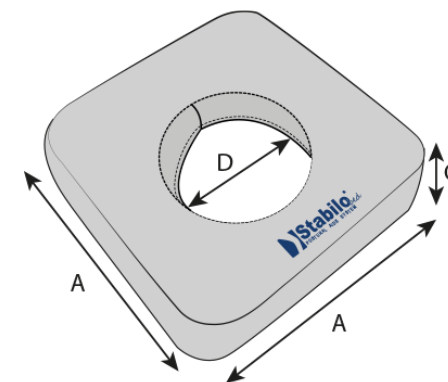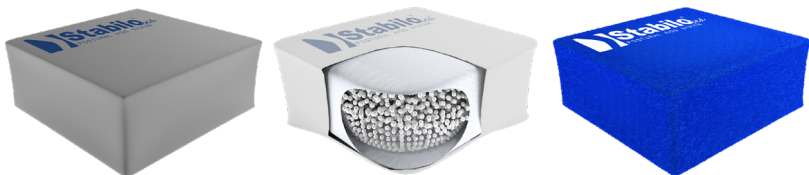
DESCRIPTION

- ✓ The reflux baby wedge has been created for infants with problems related to the gastroesophageal reflux.
- ✓ Using the wedge significantly relieves the effects of the condition by putting the little users in the proper lateral position and securing them at the same time.
- ✓ The correct position of the child prevents the backflow of the gastroesophageal content, which significantly reduces the risk of choking.

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
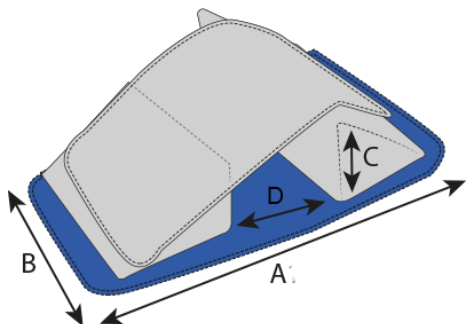
DESCRIPTION

- ✓ Bean bag P-SS-38 is a sensory floor cushion created for disabled children.
- ✓ It is great help during various forms of therapy. It can be successfully used for postural drainage or other procedures.
- ✓ Its inside is filled with high quality granulate which makes it possible to adjust the bean bag to the shape of the user easily and quickly.
- ✓ In addition, the granules provide high proprioceptive feeling and improve neuromotor functions.
- ✓ The cushion may be put into a different shape each time, so it is an excellent support while playing, learning or relaxing.

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Symbol Code	Rozmiar Size	Jednostka Unit	A	B	C
P-SS-38	1	cm	40	115	33

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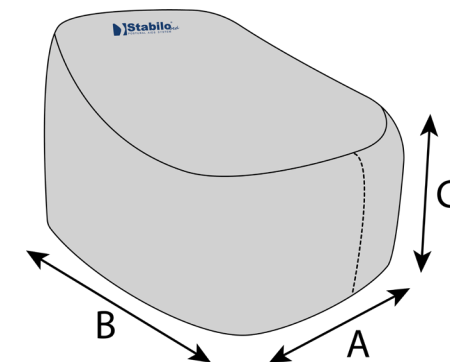
DESCRIPTION

- ✓ Rehabilitation roller is made of rigid polyurethane foam. This special-shaped device is covered by durable, washable fabric with hygienic medical certificate.
- ✓ The rehabilitation rollers are mainly used in physiotherapy exercises. A wide range of available sizes allows to use them while massage therapy or user's positioning therapy also.
- ✓ Our roller is 1st class medical device and it is easy to disinfect so it can be used by many users in health centres.

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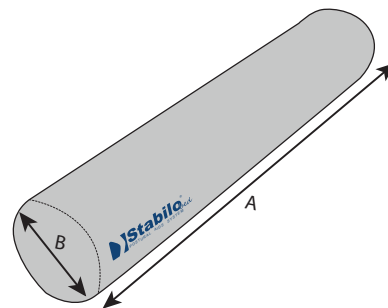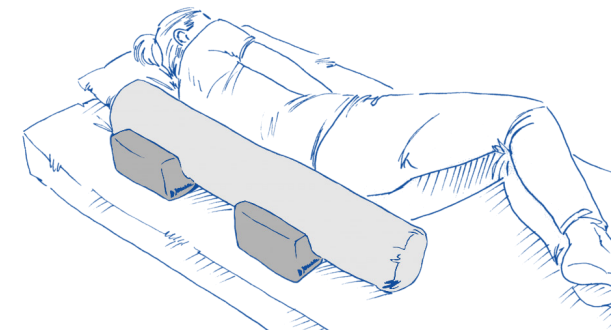
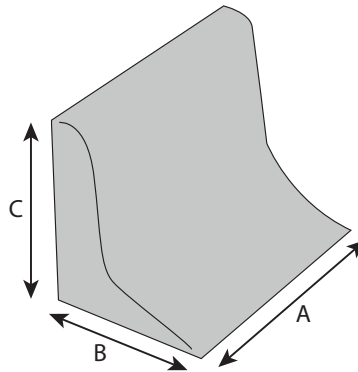
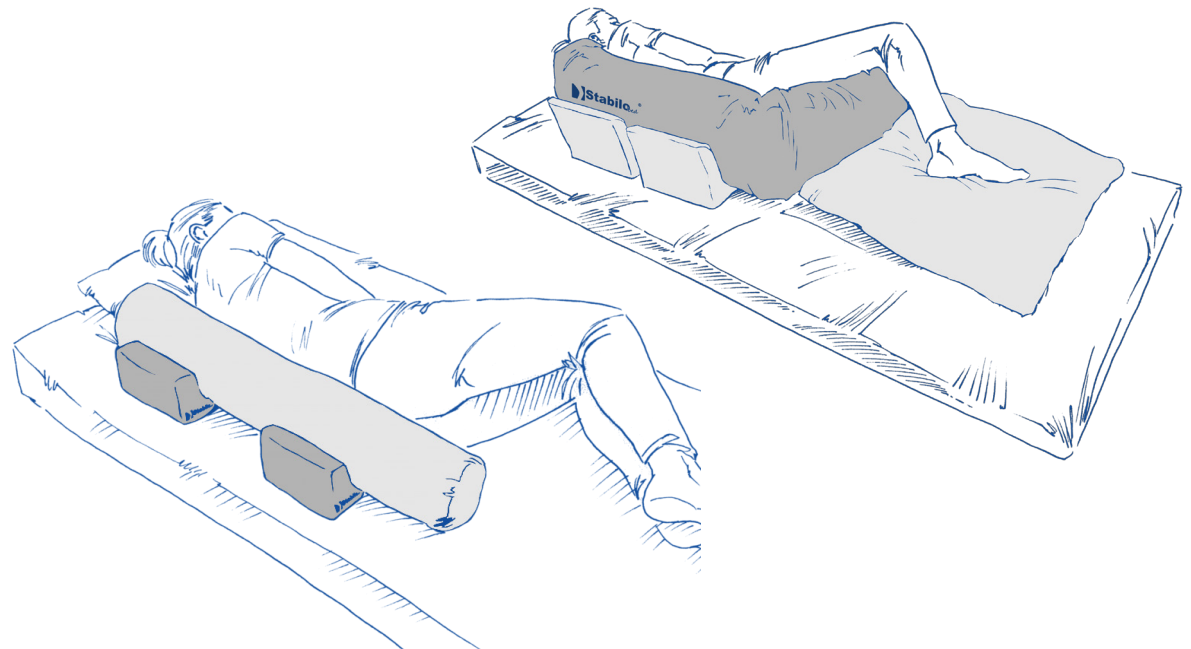
DESCRIPTION

- ✓ Thanks to the **VelcroFit™** self-adhesive function, the wedge can be attached to the bed sheet, which will prevent it from moving or slipping.
- ✓ The shape of the wedge allows for the easier and more effective use of such elements of the **STABILObed®** system as **semi-rigid bolster pillows for trunk support (P-SS-17)**.
- ✓ Highly recommended for heavy and spastic users who stay in the supine position for a long time.
- ✓ It provides the additional stabilization and comfort.

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
DESCRIPTION

P-SS-54 is an excellent addition in positional and anti-bedsores therapy. It is also an invaluable aid in daily hygiene activities. The cushion is made of a combination of soft and rigid foam, covered with an elastic PureMed™ material. It is entirely antibacterial, water-resistant and easy to disinfect, so it can be successfully used by many people, also in such places as hospitals, rehabilitation centers and nursing homes. The P-SS-54 cushion significantly facilitates turning the user from side to side. This makes caring for a bedridden person easier and does not require the assistance of additional people.

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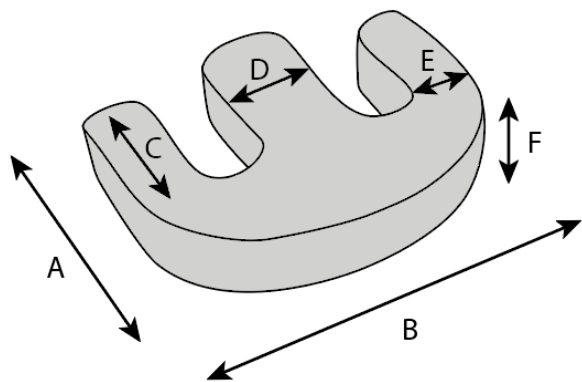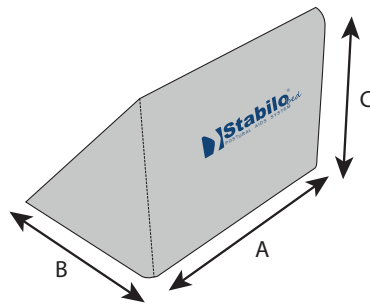
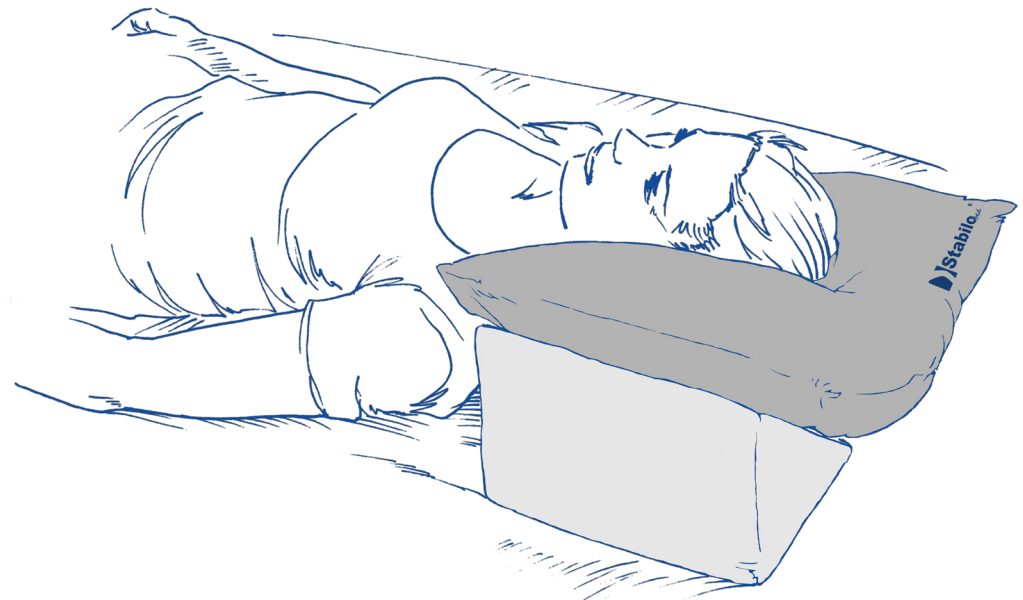
DESCRIPTION

- ✓ Thanks to the **VelcroFit™** self-adhesive function, the wedge can be attached to the bed sheet, which will prevent it from moving or slipping.
- ✓ It is highly recommended for heavy and spastic users who stay in the supine position for a long time.
- ✓ It provides the additional stabilization and comfort.

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
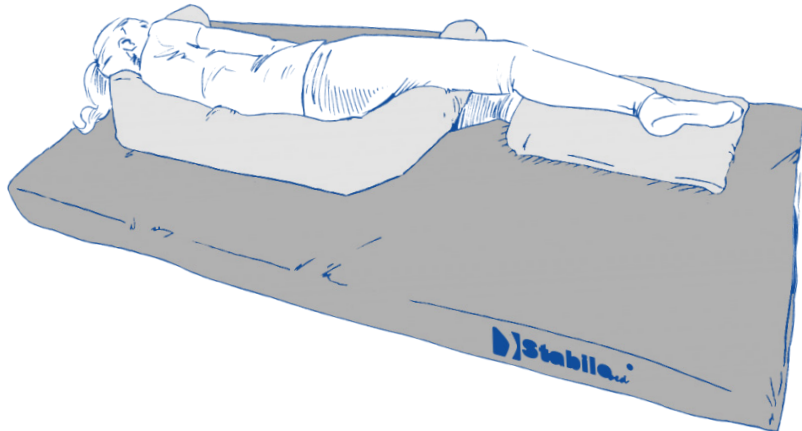
DESCRIPTION

- ✓ The sheet is made of soft velvet in white or blue colour.
- ✓ The sheet has a special foam layer which prevents the velvet from rolling on the mattress.
- ✓ The self-adhesive function makes it possible to attach the **STABILObed®** cushions to it, which prevents them from moving as most cushions have a **VelcroFit™** system.
- ✓ An elastic trimming keeps the sheet in its place.
- ✓ It is also possible to make custom-made sheets with non-standard dimensions.

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Symbol Code	Rozmiar Size	Jednostka Unit	A	B
B-SS	1	cm	90	200

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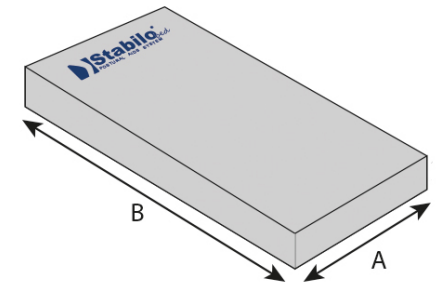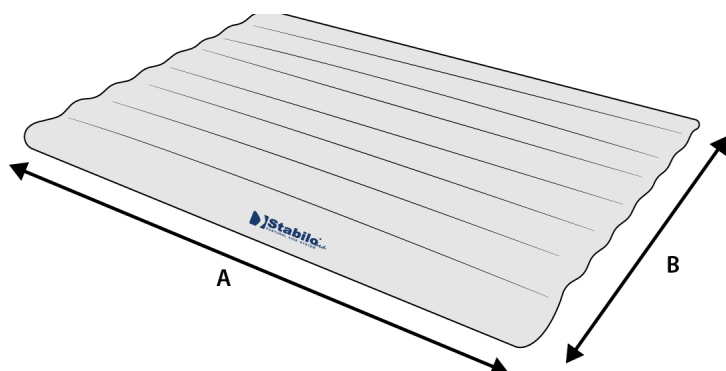
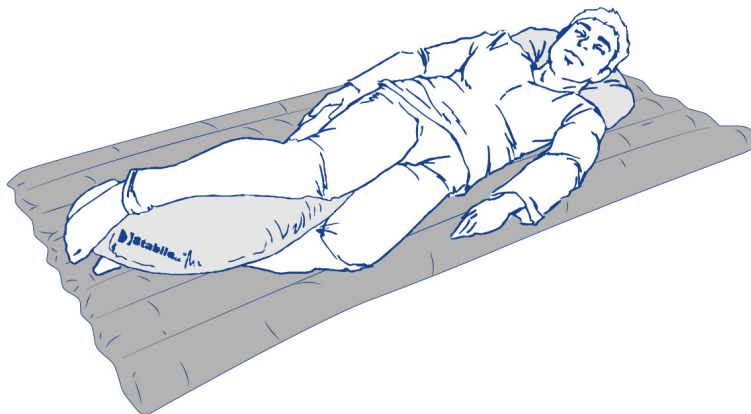
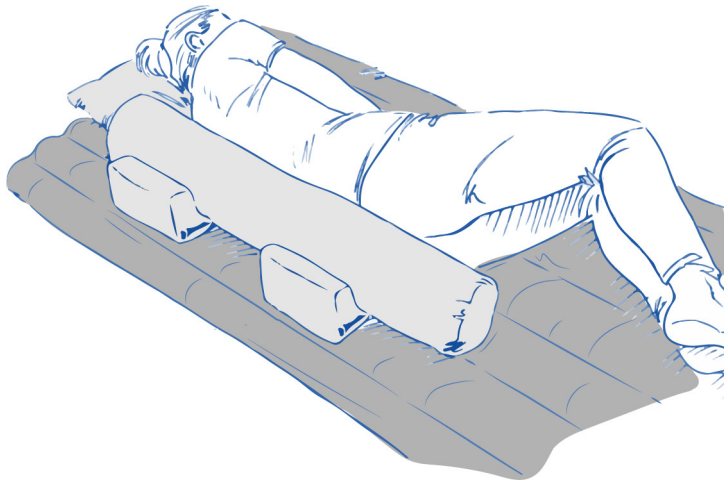
DESCRIPTION

The modular mattress enriches the **STABILObed®** range of products. The modular design based on 8 elongated segments filled with granulate is an invaluable support for the prevention of bed sores and during long-term care. Thanks to this design, **PureMed™** material and spacer fabric, the mattress cover perfectly adjusts to the shape of the user's body. The features of the cover and special openings between the segments provide a proper ventilation of the body and at the same time liquids such as sweat, blood or urine are not be able to get through the cover inside the mattress. **PureMed™** is a material that is very safe for users and also easy to disinfect. Due to the fact that it contains antibacterial components based on Silver Zeolite, **PureMed™** material is perfect for multiple use by different patients in hospitals or rehabilitation centers as well as in cases of a high risk of infection. **PureMed™** is light, soft and easy to shape. Hygienically certified by the National Institute of Public Health, it also has **OEKO-TEX® STANDARD 100®** certification which is granted to exceptionally skin-friendly parameters.

Durable and well-thought-out design allows other **STABILObed®** components to be mounted directly on the mattress. It is possible thanks to the system of Velcro straps and special openings between the segments. The arrangement of the openings on the whole mattress corresponds to the different body segments that we want to support - head, shoulders, trunk, upper limbs, lower limbs, etc. These segments also allow for complete flexibility when using other elements of the system, as needed along or across the mattress (check the correct way to attach the cushions to the mattress in the manual). Any other cushions that do not require permanent attachment to the mattress can also be used.

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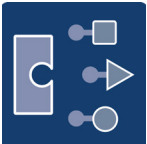
DESCRIPTION

The cushion allows the correct positioning of the lower limbs in a slight bend while lying down. The wedge protects the user from the formation of adduction contractures in the hip joints and reduces the risk of bedsores on the medial side of the lower limbs. It helps to maintain the correct supine position by controlling the legs position, preventing them from crossing. The cushion also prevents the formation of hyperextensions in the knee joints.

It is highly recommended for the bedridden patients. It can be successfully used in both the supine and semi-reclining position, e.g. in post-operative therapy.

Using the cushion reduces the overstraining of the lumbar spine, improves the circulation and reduces the risk of the swelling of the lower limbs.

Usage:

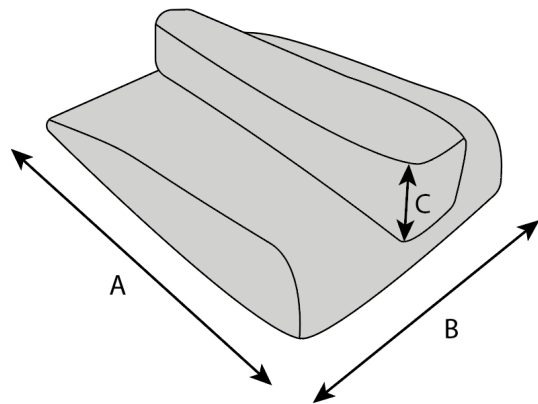
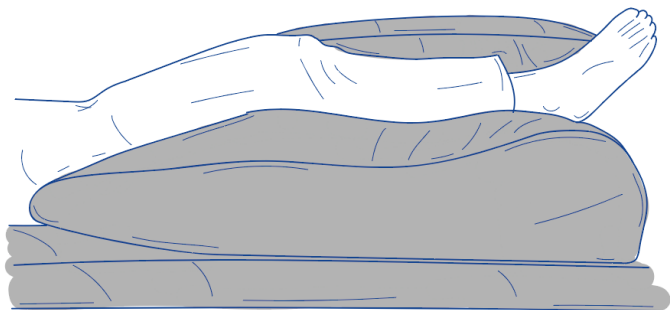
- used together with the lateral positioning cushion (**P-SS-08**) helps to put the user in the Semi-Fowler's position, where the pelvis and lower back are at **30°**.
- together with the universal pillow (**P-SS-31**) helps to put the user in the anti-bedsores position while lying on the back.

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© Sławomir Wroński 2023