

### УГОЛ СГИБАНИЯ/РАЗГИБАНИЯ ОПРЕДЕЛЯЕТ ВРАЧ ИЛИ ФИЗИОТЕРАПЕВТ

1. Установить ортез в положении желаемого ограничения сгибания или разгибания (рис. 1)
2. Вкрутить шуруп(ы) в соответствующие места обозначенные на шине (рис. 2)  
A - Ограничение угла разгибания (напр. 30° (рис. 3)  
B, - Ограничение угла сгибания (напр. 45° (рис.3)

ВНИМАНИЕ! ПЕРЕД УСТАНОВКОЙ УГЛА СГИБАНИЯ/РАЗГИБАНИЯ РЕКОМЕНДУЕТСЯ НАДЕТЬ ОРТЕЗ НА КОНЕЧНОСТЬ

Если ортез имеет 2 боковые шины, следует установить одинаковый угол на каждой шине отдельно (п. 1-2)


Шины не требуют технического обслуживания.

Срок годности: НЕ ОПРЕДЕЛЕН 2013  
Гарантийный срок: 24 МЕСЯЦА ОТ ДАТЫ ПОКУПКИ



Допуск на рынок: продукт зарегистрирован  
в Управлении по регистрации медикаментов,  
Медицинских изделий и биоцидов.  
Изделие класса I

REH4MAT Sławomir Wroński  
ul. Piaski 47, 36-060 Głogów Mlp., Poland  
tel. +48 16 621 42 20; fax +48 16 621 42 13  
e-mail: biuro@reh4mat.com, www.reh4mat.com

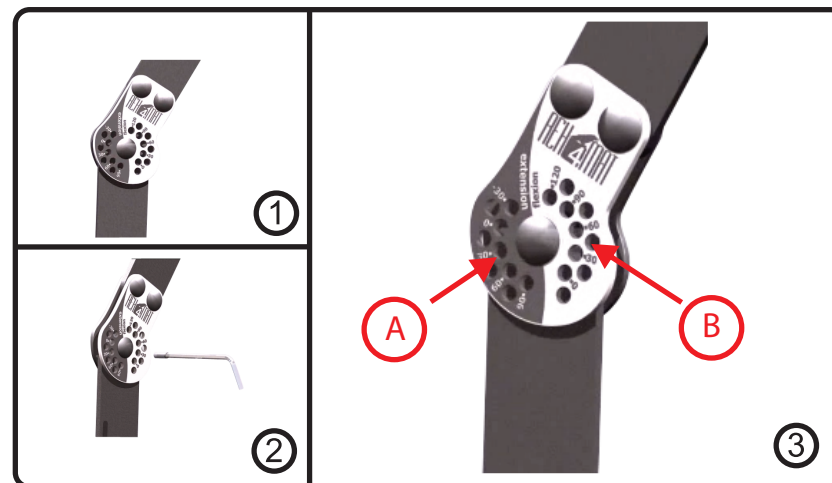

### THE ANGLE OF BENDING AND STRAIGHTENING IS SPECIFIED BY THE DOCTOR OR PHYSIOTHERAPIST

1. Place the support in the position in which you want to stabilize the flexion (extension).
2. Place the screws in the holes showed in the picture on the left.

A- restriction of the extension angle at e.g. 30 degrees (3)  
B- restriction of the flexion angle at e.g. 45 degrees (3)

#### ATTENTION:

first put the device on the limb than adjust the angle of bending and straightening.

If the orthosis is equipped with 2 side splints, adjustment is done on each side splint separately, setting the same angle of bending and straightening on both splints (point 1-3).

Side splints do not require maintenance.

Expiry date - IS NOT DETERMINED. 2013

Guarantee period - 24 months from the date of purchasing.



Admitted on the market: product registered  
in the Department of Registration of Healing Products,  
Medical Products and Biocide Products.  
Medical product first class

REH4MAT Sławomir Wroński  
ul. Piaski 47, 36-060 Głogów Mlp., Poland  
tel. +48 16 621 42 20; fax +48 16 621 42 13  
e-mail: biuro@reh4mat.com, www.reh4mat.com